
Health o meter®

Professional

OPERATING INSTRUCTIONS INSTRUCTIONS D'UTILISATION INSTRUCCIONES DE OPERACIÓN

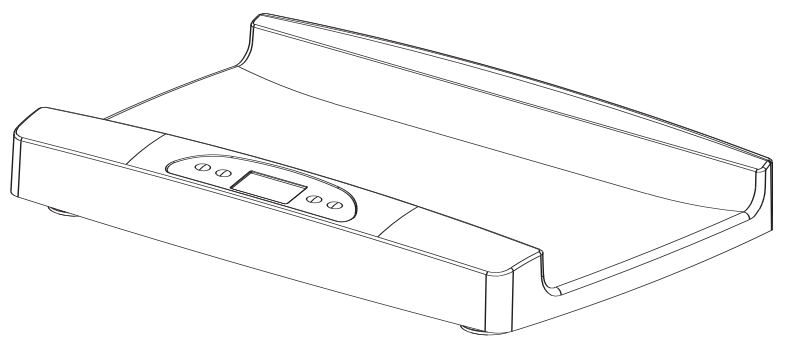DIGITAL PEDIATRIC SCALE MODEL:553KL

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DIGITAL PEDIATRIC SCALE MODEL:553KL

IMPORTANT

- To prevent falls or injury, place the scale securely far from the edge of the counter or table that is used to hold the scale.

USING THE SCALE

- 1: Install 6 new AA alkaline batteries (included) into the battery compartment underneath the scale base.

- 2: Place the scale on a hard, flat surface for safety and accuracy.
- 3: Press the ON/OFF button to start the scale. 88:8.8 and then 0:0.0 will appear on the display, along with LB and OZ underneath (or 0.00 Kg). To change the units to Kg, press the KG/LB button once.*

- 4: Lay the baby on the scale so that its weight is evenly distributed on the tray.
- 5: When a stable weight is calculated a small ring will appear on the display, along with the baby's weight. Depending upon the units selected, the weight is displayed in pounds and ounces (LB) or in decimals (Kg).
- 6: If the display shows "OVER" you have exceeded the capacity of the scale.
- 7: For your convenience, a measuring tape is included with the 553KL scale. You may affix the measuring tape to the bottom or side of the weighing tray.

- *To change the scale default units to kilograms, or to deactivate the KG/LB button, please contact Technical Support at 1-800-638-3722 or 1-708-598-9100.

HOLD/RELEASE FUNCTION

The HOLD function allows you to read the weight after removing the baby from the scale platform.

- 1: Once a stable weight is displayed (ring appears in upper left hand corner of display) press the HOLD/RELEASE key.

- 2: "HOLD" will appear in the lower left hand corner of the display.
- 3: Remove the baby from the weighing tray. The scale will continue to display the weight.
- 4: Press the HOLD/RELEASE key again to return the display to 0.

TO TURN OFF THE SCALE

- 1: The scale will shut off automatically after 2 minutes of inactivity.
- 2: To manually turn off the scale, press the ON/OFF key.

DEACTIVATE AUTO-OFF

- 1: With the scale turned off, press and hold the HOLD/RELEASE key.
- 2: Press and release the ON/OFF key while still holding the HOLD/RELEASE.
- 3: Release both keys and AOF=0 will be displayed to confirm that the Auto-Off feature has been deactivated.
- 4: To reactivate the Auto-Off feature, turn scale off and repeat steps 1 through 3. AOF=1 will be displayed when Auto-Off is activated.

INSTALLING BATTERIES

Use only AA alkaline batteries.

- 1: The battery compartment is located underneath the scale base.
- 2: Place 6 new AA batteries into battery compartment. Make sure the '+', '-' and '-' are positioned correctly, as shown on the bottom of the battery compartment.
- 3: Replace battery cover.
- 4: 120V AC adapter available separately (model# ADPT13).

Replace the batteries when the display reads 'LoBat'.

ZERO/CLEAR (TARE) FUNCTION

The Tare Function allows you to place other items on the scale while weighing the baby without affecting the baby's weight reading.

- 1: Weigh the item you would like to tare.
- 2: Press and release the ZERO/CLEAR key to change the weight to 0.
- 3: Place the baby on the scale's tray and weigh normally.

NOTE: If the scale is empty and the display does not read 0, press the ZERO/CLEAR key once to return the display to zero.

MAINTENANCE

- 1: To clean the scale, wipe the outside of the scale with a soft cloth and mild detergent.
- 2: Do not use abrasive cleaners. Never place the scale in water or spray with liquid.
- 3: Do not store scale near chemicals, heaters or radiators.
- 4: Do not store scale on its side.
- 5: This scale is an extremely sensitive weight device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not place anything on the scale.

ONE YEAR LIMITED WARRANTY

Not legal for trade.

LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

Call 1-800-638-3722 to get Warranty Service.

Another innovation from the people at

Health o meter®
Professional

7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615
www.healthometermedical.com

Health o meter® Professional products are manufactured, designed and owned by Pelstar LLC. Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

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