seca cardioteest 100

Bedienungsanleitung
Operating instructions
Mode d'emploi
Manual de instrucciones
Istruzioni per l'uso
Betjeningsvejledning
Bruksanvisning
Bruksanvisning
Käyttöohje
Bedieningshandleiding
Instruções de uso
Οδηγίες χειρισμού

Deutsch	Seite 3
English	Page 3
Français	Page 9
Español	Página 9
Italiano	Pagina 15
Dansk	Side 15
Svenska	Sida 21
Norsk	Side 21
Suomi	Sivu 27
Nederlands	Pagina 27
Português	Página 33
Ελληνικά	Σελίδα 33

Herzlichen Glückwunsch

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Congratulations

In the seca cardioteest 100 ergometer you have acquired a highly-accurate and well-designed machine.

seca has been putting its experience at the service of health for over 150 years and as market leader in many countries of the world, is always setting new standards with its innovative developments in weighing and measuring.

The seca cardioteest 100 ergometer (Model 545) is an electromechanically-controlled, speed-independent pedal ergometer to allow patients to expend effort while seated.

It is intended for use in the medical field under the supervision of medical specialists.

It is furthermore designed both for independent operation (rehabilitation measures) and for applications in conjunction with electrocardiographs (ECG after effort). For this purpose, it has a control panel on the handlebars with a digital display and facilities for connecting an ear-clip/ECG plotter.

The seca cardioteest 100 ergometer has national design type approval and is mains-operated.

Qualified diagnoses through practical ergometry

The seca cardioteest 100 meets all the requirements of a modern ergometer as per DIN standards 32932 and 13405:

- incremental increase in effort to push test subject to the limit
- actual pedalling effort at all speeds matches effort set
- simple to use
- continuous pulse rate monitoring by ECG unit or ear-clip (available as an accessory)
- automatic zero reset before every test
- electronics self-test

Safety

Before using the new ergometer, please take a little time to read the following safety instructions.

- Follow the operating instructions.
- When setting up the machine, please ensure that it is steady and that there is adequate space to get on and off the ergometer.
- Route the mains cable and any other connections so that there is no risk of people tripping.
- Avoid setting up the machine in places where severe electromagnetic radiation may occur (e.g. next to X-ray machines, radio transmitters etc.) and avoid using radio telephones in the immediate vicinity of the machine.
- Please check before connecting the machine whether the alternating voltage given on the rating plate is the same as local mains voltage, shown on the electricity meter.
- Have repairs carried out exclusively by authorised persons.
- Have servicing carried out regularly (see chapter entitled "Servicing").

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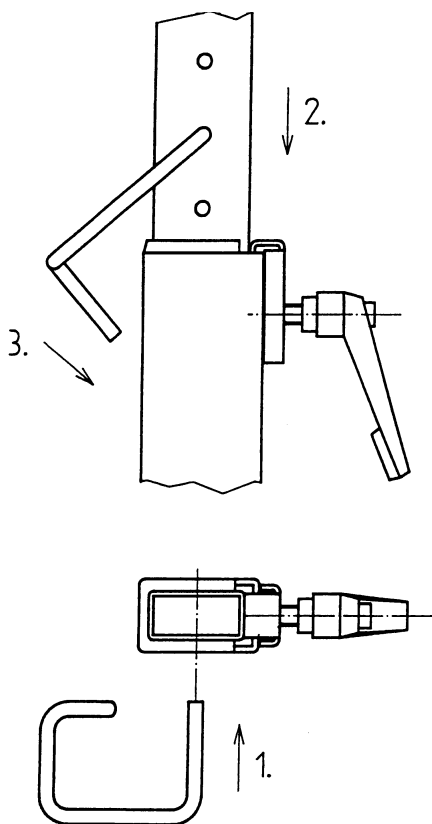
Before you really get going...

Setting up the machine

Connect the mains unit. Route the mains cable so that it is not in anyone's way and cannot be tripped over.

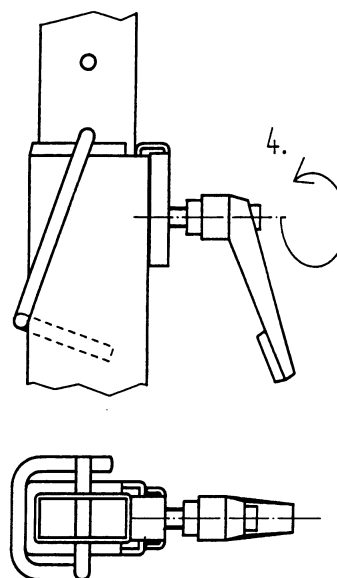
Attach the control panel to the handlebars so that the display faces you. Only then will the additional speed display be visible to the test subject. Check that the displays are easy to read and make sure that the display windows are not partially obscured by other objects. This will prevent potential misreadings. Connect the cable of the control panel to socket 4.

If your seca cardiostest 100 is equipped with an ear-clip to measure pulse rate, connect this to socket 5.

If you want to use the ergometer in combination with an ECG plotter to record an ECG after effort, connect the ECG unit (via adapter model 462) to socket 5 of the seca cardiostest 100 (in this case, the ear-clip cannot be used). seca Customer Service will be glad to provide you with more information about this.

Preparing the test

Put the saddle (cf. diagram below) and handlebars at the right level for the patient. (The saddle is in the right place if the patient's legs are at a slight angle with the pedal at its lowest point. The handlebars should be at a level where the patient is sitting comfortably with the upper body slightly curved).

1. Insert fixing pin in saddle support.
2. Push in saddle support until fixing pin abuts.
3. Secure fixing pin, i.e. abut against support tube.
4. Slightly tighten up clamping lever.

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After the ECG plotter, switch on the ergometer at the main switch. The display and electronics can be tested before the start of the test by pressing the Test key. All the LEDs are switched on for two seconds and off for two seconds. 1.8.8. appears in the Speed display and Pulse rate display, then the test value (a number between 97 and 103) appears in the Effort display.

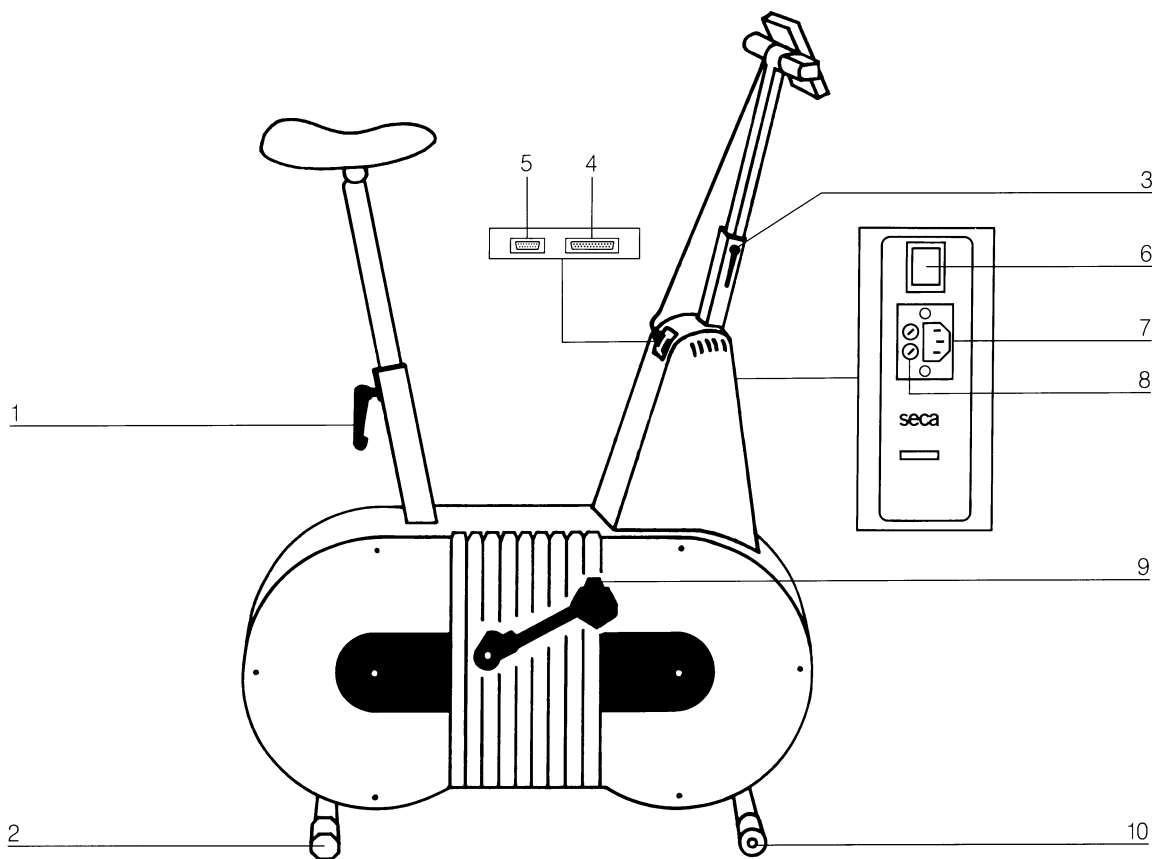
If appropriate, attach the ear-clip to the ear-lobe and check whether the pulse lamp is flashing regularly. It may be necessary to rub the ear-lobe vigorously once and to check that the ear-clip is properly in position.

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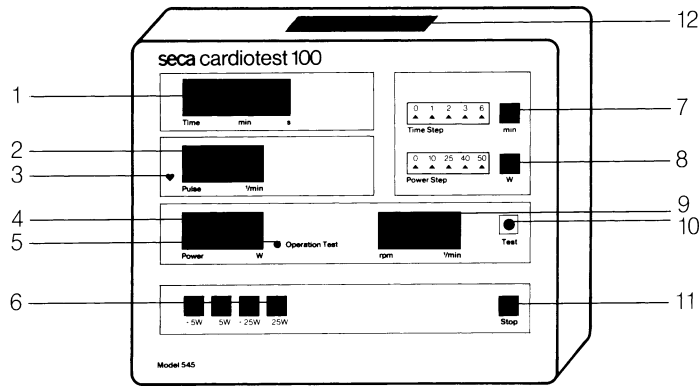
- 1 Clamping lever for saddle height adjustment
- 2 Adjustment flaps to compensate for uneven
floors
- 3 Clamping lever for handlebar height adjustment
- 4 25-pin socket for cockpit
- 5 15-pin socket for ear-clip (available as an
accessory) or for ECG plotter
- 6 Main switch (On/Off)
- 7 Socket for mains connection
- 8 Fuse
- 9 Foot straps to keep feet securely on pedals
- 10 Castors to make the machine easy to move

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- 1 Time display
- 2 Pulse rate display
- 3 Pulse lamp (flashes regularly when pulse rate recording properly, only when ear-clip used)
- 4 Effort display
- 5 Luminous spot for test value display
- 6 Keys to change effort
- 7 Key to select time levels
- 8 Key to select effort levels
- 9 Speed display
- 10 Test key
- 11 Key for ending program
- 12 Additional speed display for test subject

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Effort programs

Manual effort

Use the Change effort keys to enter the desired initial effort.

The seca cardioteest 100 electronically keeps the effort set by you constant, regardless of the speed at which the test subject pedals.

During the test, you can increase or reduce effort at will in 5 or 25 W increments using the Change effort keys.

When the test is complete, press the Stop key.

Programmed load

Enter the desired starting effort using the Change effort keys.

Using the key to select time levels, define how long an effort level is to be maintained.

Using the key to select effort levels, determine by how many watts effort is to be increased once the previously-defined time level has elapsed.

As soon as the patient starts to pedal, the effort program runs according to your specifications.

When the test is complete, press the Stop key.

Cleaning

Clean the machine with a damp cloth, using a commercially-available domestic cleaning agent or disinfectant. Please ensure that no fluids get into the machine.

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What do I do if ...

- ... 5 L-P appears in the Effort display when the Test key is pressed?
 - Allow machine to come to a standstill for automatic calibration.
- ... not all the LEDs come on when the Test key is pressed?
 - Inform seca Service.
- ... a value other than a number between 97 and 103 appears in the Effort display as a test value?
 - Inform seca Service.
- ... the pulse lamp does not flash regularly?
 - if using an ECG plotter: check connection to ECG plotter.
 - if using ear-clips: rub ear lobe vigorously again, check ear-clip properly in position.

Disposal

If it is no longer possible to use the ergometer, the responsible waste disposal association will be glad to provide information on the measures necessary to dispose of it properly.

Servicing

Medical pedal ergometers are subject to servicing regulations which provide for the ergometer being checked every 2 years. Please check the date for the next service on the sticker attached to the machine. Servicing may only be performed by an authorised service agency. Please contact seca Customer Service.

Technical data

- infinite seat height adjustment (34 cm)
- infinite handlebar height adjustment (34 cm)
- adjustment flaps to compensate for uneven floors
- two castors at front to make machine easy to move
- facility for recording pulse rate by ECG plotter (with cable adapter) or by ear-clip as an accessory
- digital display field for effort, speed, pulse rate and time
- height of digits: 13 mm
- electronics self-test at the push of a button
- serial interface
- electronically-controlled eddy-current brake up to 400 W, speed-independent
- microprocessor control
- standardised effort range as per DIN 13405: 25 W to 400 W, error limit +/- 5 %, but not greater than +/- 3 W
- standardised speed range as per DIN 13405: 30 - 90 rpm, error limit +/- 2 rpm
- braking torque range: 1 - 129 Nm
- moment of inertia of disc flywheel: 10.6 kg/m²
- crank radius of pedals: r = 170 mm
- alternating current 50/60 Hz
- power consumption: max. 100 VA
- protection class 1/Type B (IEC 601-1)
- Medical product in accordance with Directive 93/42/EEC: Class I with measuring function

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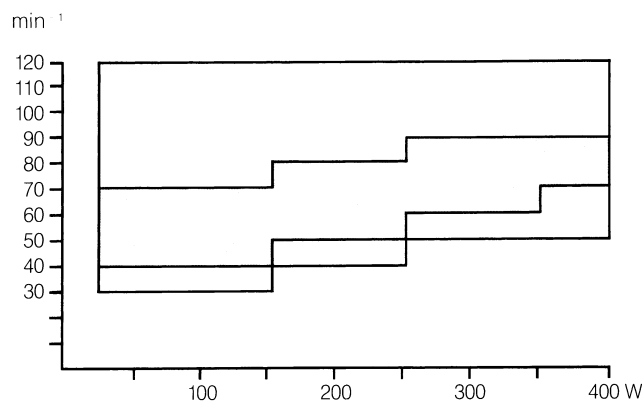
Rpm - effort range

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as per the specifications of the German Institute of Physics and Technology

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Accessories

- adapter for connecting ECG plotter
- ear-clip for recording pulse rate

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Guarantee period

We give a guarantee of 1 year from date of purchase for defects attributable to poor materials or workmanship. Defects which come under guarantee are repaired free of charge. Other claims cannot be taken into account. Transport to and from ourselves will be at the customer's expense if the machine is located somewhere other than the purchaser's place of residence. The guarantee is invalidated if the machine not handled properly. We would ask our customers abroad to contact the sales organisation of the appropriate country in the event of a claim under guarantee.

Important: guarantee valid only in combination with proof of purchase!

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Vogel & Halke GmbH & Co